

UltraMax **Gemini**TM

2-Piece Disposable Adhesive Ostomy System
with "E-Z Guide"TM Locating System



MARLEN

*Available in Drainable, Urostomy
and Closed End Pouches*

The Complete UltraMax™ Gemini™ System Offers Unparalleled Choice.

3 Pouch Options



All pouches available in Opaque with Kwick-View™ “peek-a-boo” feature (see opposite page) or Transparent.



UltraMax™ Gemini™ Closed End

- For colostomies
- “Flo-Thru™” Charcoal Filter system continuously filters odors from both sides of the filter thanks to its patented central suspension design

UltraMax™ Gemini™ Drainable

- For ileostomies and colostomies
- Features MARLEN Kwick-Klose™ Fastener
- Extra-large drainage spout for easy emptying
- Available with or without filter

UltraMax™ Gemini™ Urostomy

- For urostomies
- “Free-Flow” stem allows unobstructed flow of urine into leg or night bags, even if pouch twists
- “Push-Pull” E-Z Drain™ Valve is permanently sealed to the pouch – no wet fingers when emptying pouch
- Non-return reflux valve

MARLEN UltraMax™ Gemini™ Features

- All the advantages of a 2-piece system
- Maximum flexibility
- Maximum comfort
- Maximum security
- Odor-proof material
- Quiet, built-in comfort cover on body side
- Leak-proof for added protection and security
- Available in either transparent material, or opaque with Kwick-View™ “peek-a-boo” feature

Marlen’s Proprietary E-Z Guide™ Locating System

... the easiest method yet for properly locating the pouch onto the flange



1. Simply slide the belt tab on the pouch into the “E-Z Guide” nest on the back side of the body flange.



2. Carefully roll the pouch sideways over the locating ring on the back side of the flange.



3. Secure opposite-end belt tab into the other “E-Z Guide” nest, thereby securing the pouch to the flange.



4. Press the pouch to the flange in a circular motion to complete the adhesive bond.



All MARLEN UltraMax™ Gemini™ products are ultra-flexible and easily conform to the contours of the body.


2 Flange Styles...4 Flange Options

Flat – Cut-to-Fit
(with a 12 mm starter hole)

Flat – Pre-Cut
(with stoma openings from 12 mm to 55 mm)

Shallow Convexity – Cut-to-Fit
(with a 12 mm starter hole)

Shallow Convexity – Pre-Cut
(with stoma openings from 12 mm to 38 mm)

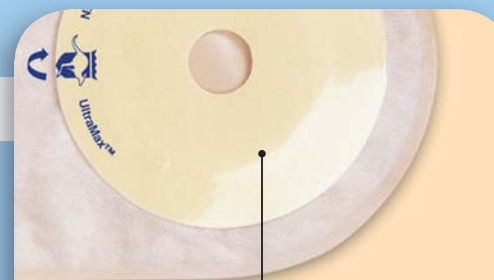

AquaTack™ Hydrocolloid Barrier

- Covers entire flange for maximum skin protection
- Absorbs skin moisture
- Adheres to moist skin
- Controlled adhesion (seals securely and removes easily from sensitive skin)

Introducing Kwick-View™ from MARLEN


Marlen’s Kwick-View™ “peek-a-boo” feature is a hybrid solution that allows patients to have the discreet confidence that comes with an opaque pouch along with the quick accessibility and vision that comes with a transparent pouch.

- Check stoma without removing the pouch
- Ideal for post-operative applications
- Available on MARLEN UltraMax™ Gemini™ two-piece disposable drainable systems




UltraMax™ Gemini™ Specifications


*UltraMax™ Gemini™
Closed End*

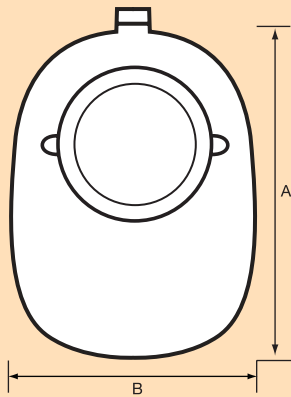

*UltraMax™ Gemini™
Drainable*

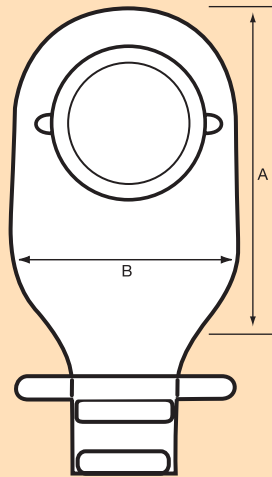

*UltraMax™ Gemini™
Urostomy*

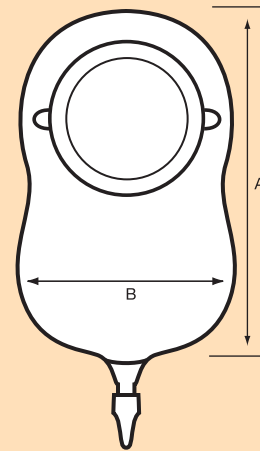

Length A	Width B	Capacity (approx.)
7-1/2" (19 cm)	5-1/2" (14 cm)	9 oz. (270 cc)

Length A	Width B	Capacity (approx.)
9" (22.9 cm)	5-3/4" (14.6 cm)	16 oz. (480 cc)

Length A	Width B	Capacity (approx.)
8-3/4" (22.2 cm)	5" (12.7 cm)	14 oz. (420 cc)

*MARLEN UltraMax™ Gemini™
Elastic Waist Belt*



For more information, contact:



Marlen Manufacturing & Development Company

5150 Richmond Road • Bedford, OH 44146 USA

Phone: 216.292.7060 • Fax: 216.292.9196

E-mail: info@marlenmfg.com • www.marlenmfg.com